


WaveBP V3.0 ^{USB} Cable Link to BPM with PC

1. Make sure following points firstly

Before transmitting measured data to your computer, please make sure following points firstly:

- 1.1 Make sure Your PC with CD-ROM and USB Port for setup.
- 1.2 Insert CD into CD tray and auto run.
- 1.3 Install USB to COM Driver first.
- 1.4 Connect ear plug to the PC jack of side of the main unit, then connect USB connector to your PC. (Please see following Fig)
- 1.5 Check USB to COM Tester, and make sure the Computer COM Port is assigned for USB -> COM Port "COM1" or 2,3,4,5,6,7,8" works ok.
- 1.6 Install enclosed "WaveBP V3.0" software is compatible with "WINDOWS 98", "WINDOWS 2000" or "WINDOWS XP" system.

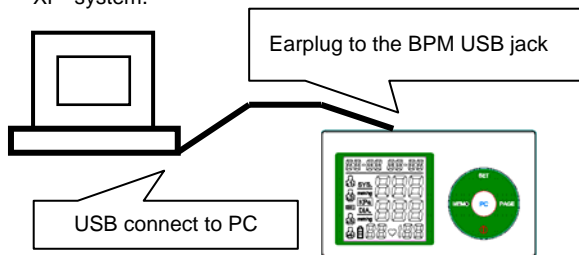

2. How to install "WaveBP V3.0" software

- 2.3 Click "Install", then you can follow the installing procedures as indicated.
- 2.4 When "WaveBP V3.0 Setup" was completed successfully" show up, press "OK", then the installing procedures are completely.

After finishing set up procedures, please click "WaveBP V3.0". The diagram will ask to input the password, please key in "medit" and click "OK" into the system.

After entering main screen, enter "Data" category and click "Communication". Then select "COM1" or "2,3,4,5,6,7,8" into "Port" category. This selection has to match connecting port as connected. After that, click "OK" to complete the preparation of transmission.

3. How to transmit data to PC

There are two options for transmitting

- Synchronized input
- Group input

3.1 SYNCHRONIZED INPUT

- 3.1.1 Turn on the computer → start → program → "WaveBP V3.0" → enter password "medit" → main screen.
- 3.1.2 Click "Data" → Read Data → "must open file to read BP data ! File not ready" will show up. → Click "OK".
- 3.1.3 Key in file name, such as "BP01", then click "Open file", then "C:\program files\WaveBP\BP01. wbp is new file. Create now ?" Will show up, and then Click "Yes".
- 3.1.4 Start measuring, and blood pressure data will be transmitted to computer immediately once measurement is done, data will transmit to PC.
- 3.1.5 Click "Exit /Save", then "Save Data to file C : \ program files\ WaveBP\ BP01. wbp ?" Will show up. At last, click "Yes"; the measurement value will be transmitted to PC successfully.

3.2 GROUP-INPUT

- 3.2.1 Turn on the computer → start → program "WaveBP V3.0" enter password "medit" → main screen.
- 3.2.2 Click "Data" → Read data → "must open file to read BP data ! File not ready" show up. → Click "OK".
- 3.2.3 Key in file name, such as "BP01", then click "Open file", then "C : \program files\WaveBP\BP01. wbp is new file. Create now ?" Will show up.
- 3.2.4 Click "OK"
- 3.2.5 Switch off the BPM and then power on.
- 3.2.6 Press " " button. The "PC" message will come up on the LCD screen. The PC receives the data automatically.
- 3.2.7 Click "Exit/Save" button.
- 3.2.8 Screen will show "save data to file C:\program files\ WaveBP \ BP01. wbp ?".
- 3.2.9 Click "OK".
- 3.2.10 BP data table will show up after transmitting.
- 3.2.11 Click "Line Diagraph " or "Bar Chart" or "Statistic diagraph" analyzing graph as you desire.

4. How to change password

If you want to change password, please click "Change password" in the "Security" category.

5. How to print blood pressure data

If you want to print it out, please click "Print" icon

